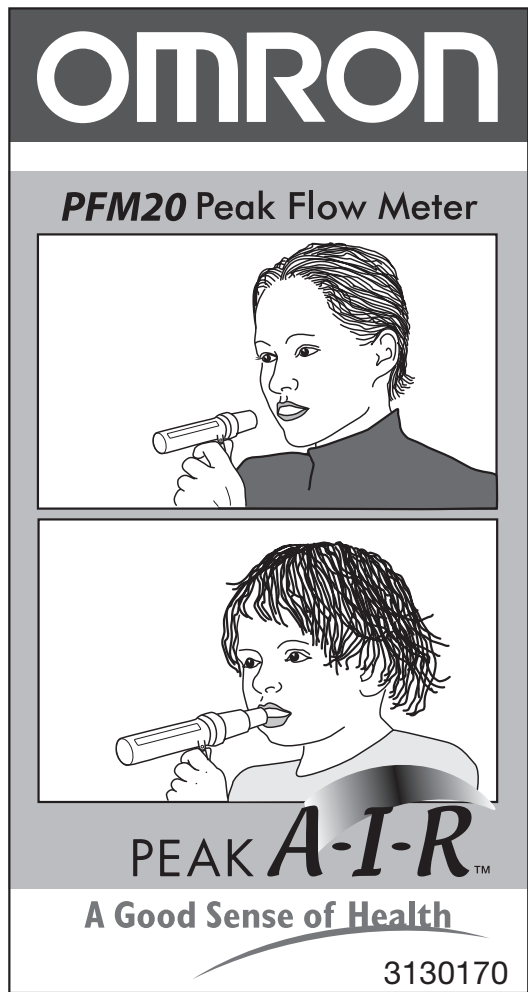
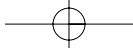
CARE AND USE OF YOUR PEAK FLOW METER

It is important to keep your Peak Flow Meter clean when in use. It should be cleaned at least once every six months and thoroughly if it has not been used for some time, fungal growths may otherwise occur. The Mouthpiece is detachable and should be disinfected before each use using a proprietary disinfectant solution such as Dettol. The OMRON PFM20 Peak Flow Meter is designed for single patient use.

Technical Information The OMRON PFM20 Peak Flow Meter conforms to ISO 23747 and the NAEPP Guidelines. Accuracy: Better than 10% or 100/ml. Readings expressed as BTPS values. Maximum resistance to flow 0.35 kPa/l/s. Store in a dry place.

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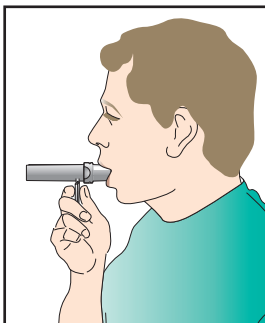
3 Stand up, if possible. Take a deep breath, place the Peak Flow Meter in the mouth and hold horizontally, closing the lips around the mouthpiece, then blow as hard and as fast as you can. Coughing and spitting into the meter will adversely affect your reading and should be avoided.

4 Note the number on the scale indicated by the pointer. If unusual readings are obtained and the meter does not appear broken - seek advice from your healthcare provider.

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